

Green Card T Air Removal & **STEAM** Penetration Test

BD 120

The Green Card T is for daily monitoring of small pre-vacuum steam sterilizers operating at 134°C for 3.5 minutes to detect the presence of residual air.



Key Features

- Complete color change - 3.5 minutes at 134°C
- Sterilization pouch protects the indicator from excessively wet conditions
- Lead free and no toxic heavy metals

Performance Characteristics

If the sterilizer is functioning properly, the indicator graphic will show a color change from purple to a uniform green throughout the entire indicator image. When excessive residual air or non-condensable gases are present (major failure), the indicator will not be a uniform green color but remain purple in the center or a shade of gray.

Physical Characteristics

- Dimensions: L 86mm x W 86mm

Storage

The storage conditions are printed on the product packaging. Protect from moisture and humidity. Store in resealable bag provided.

Shelf Life

24 months from date of manufacture. The expiry date is printed on the product packaging.

Standards Classification

ISO 11140-1 Type 2 indicators for use in specific tests and ISO 11140-6 sub-atmospheric cycle

SteriTec Part No.

BD 120, Green Card T
Air Removal & Steam
Penetration Test

Packaging

15 bags of 15
tests + 1 ring

FTP-168 Rev. I