


# Green Card S/T Bowie-Dick Test

## BD 117, BD 118

The Green Card S/T Bowie-Dick Test is for daily monitoring of pre-vacuum steam sterilizers operating at 134°C for 3.5 minutes.

The Green Card S/T is designed to operate in sterilizers with subatmospheric or trans-atmospheric cycles as defined in ISO 11140-4.

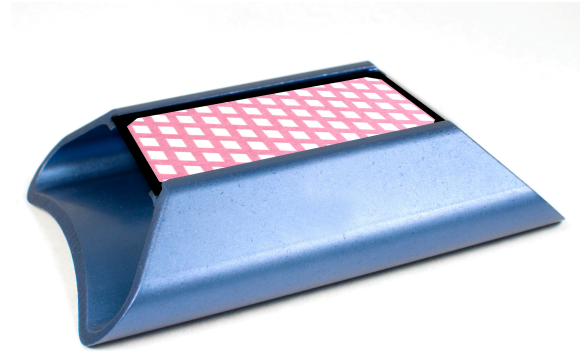

### Key Features

- Complete color change - 3.5 minutes at 134°C
- Re-usable holder keep card in place
- Tests for air leaks, wet steam, superheated steam and non-condensable gases
- No disposable paper pack
- Lead and toxic heavy metal free

### Performance Characteristics

If the sterilizer is functioning properly, the indicator graphic will show a color change from purple to a uniform green throughout the entire indicator image. When excessive residual air or non-condensable gases are present (major failure), the indicator will not be a uniform green color but remain purple in the center or a shade of gray.

### Physical Characteristics

- Dimensions Card: L 90mm x W 46mm

### Storage

The storage conditions are printed on the product packaging. Protect from moisture and humidity. Store in resealable bag provided.

### Shelf Life

24 months from date of manufacture. The expiry date is printed on the product packaging.

### Standards Classification

ISO 11140-1 Type 2 Indicators for use in specific tests; ISO 11140-4

#### SteriTec Part No.

BD 117, Green Card S/T  
Bowie-Dick Test

BD 118, Green Card S/T  
Starter Kit

#### Packaging

8 cases of 4 bags  
of 30 tests

8 kits of 31 tests + 1  
holder