

Battery Maintenance Required

This message will be displayed if a battery installed in the IABP requires maintenance and conditioning.

- **The Cardiosave IABP can still be used.**
- **Continue operation on AC power.**
- **Monitor the system closely for a Low Battery alarm.**
 - If battery operation is required, the battery run time may be reduced.
 - If a Low Battery alarm occurs, immediately connect the system to AC power or install a fully charged battery.

Additional Information:

- Execute the Battery Maintenance as soon as possible.
- The battery does not require replacement.
- If at any time a battery requiring maintenance is removed from an IABP console, place the battery inside a battery transport storage case and deliver to biomedical or service department. Should any batteries requiring maintenance remain in the IABP once therapy has ended, label the IABP as requiring battery maintenance and deliver to biomedical or service department.

GETINGE 



Battery Replacement Required

This message will be displayed if a battery installed in the IABP has become unreliable and requires replacement.

- **The Cardiosave IABP can still be used.**
- **Continue operation on AC power.**
- **Replace with a new battery.**
 - If new battery is unavailable and battery operation on the indicated battery is required, the battery run time will be reduced.
- **Monitor for Low Battery Alarm.**
 - If a Low Battery alarm occurs, immediately replace (or swap) the battery or return to AC power.

Additional Information:

- Execute the Battery Maintenance as soon as possible.
- The battery does not require replacement.
- If at any time a battery identified as requiring replacement is removed from the IABP, place the battery inside a battery transport storage case and deliver to biomedical or service department. Should any batteries requiring replacement remain in the IABP once therapy has ended, label the IABP as requiring battery replacement and deliver to biomedical or service department.

GETINGE  

