

# Getinge Clean

## Manual Detergent High Foam

Manual Detergent High Foam is a high quality, liquid formulation designed to be used by hospitals and laboratories.



### Application

For manual prewashing of surgical instruments or equipment. Not for machine use.

### Key Features

- Neutral pH, mild to skin
- Readily enters solution and completely dissolves
- Easily penetrates organic soils
- Combination of anionic and nonionic detergents, which emulsifies blood and protein for quick removal
- Concentrated, economical use dilution ratio
- Free rinsing for film-free results
- Noncorrosive; safe on stainless steel, aluminum, gold and other soft metals
- Hard water tolerant, effective in all water conditions
- Biodegradable

**Quality Statement:** Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

## Usage Information

Water Hardness (Grains Per Gallon)	Product Dosage	
	(Oz./Gal.)	(mL/L)
0 – 5	0.2	1.6
6 – 9	0.4	3.1
10 – 12	0.6	4.7
13+	1.0	7.8

Dilute product with warm water as shown in chart above. Soak instruments or utensils for 5–10 minutes. Drain and rinse. Process through instrument washer. Wear rubber gloves and safety glasses for prolonged contact. If you have additional questions, consult your Getinge representative.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

## Physical and Chemical Properties

- Clear pink, viscous liquid
- Mild, floral fragrance
- pH value: 8.0–8.5

NOTICE: For institutional and industrial use only. Not for home use. Refer to SDS for more information regarding safety, handling and storage and waste disposal methods.

---

## Ordering Information

Getinge Clean	Order Number
4 x 1 Gallon	61301605271

---

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2017 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044128 REVA

**Manufacturer:** Intercon Chemical Co, 1100 Central Industrial Dr., St. Louis, MO 63110  
**Sales Office:** Getinge, 1777 East Henrietta Road, Rochester, NY 14623

[www.getinge.com](http://www.getinge.com)

**GETINGE** 