

Green Tape International

CI 159/.75/INT & CI 159/1/INT

Indications for Use

SteriTec Green Tape can be used in 121°C gravity sterilizers and 134°C pre-vacuum sterilizers. Stripes turn black after exposure to steam sterilization conditions, thus providing identification of processed items. This product contains dry natural rubber.

Color Change

Indicator Stripes: Light green to black

Instructions for Use

1. Load the sterilizer as normal.
2. Tape the load.
3. Process the load.
4. Unload the sterilizer.
5. Inspect the tape for color change.

Color Change

Based upon conditions in the sterilizer, the color change on stripes is from light green to black if exposed to a steam process. If the color change on the stripes has not sufficiently changed then resterilization is recommended.

Storage

Store at room temperature 10° - 38°C, between 10-60% Relative Humidity and away from light sources.

Expiry Date

The expiry date is printed on the product packaging. Do not use after the expiration date on the product packaging.

Interfering Substances or Conditions

There are no known interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

Release of Toxic Substances

The indicator releases no known toxic substances in sufficient quantities to cause either a health hazard, or a hazard to the intended properties of the product being sterilized, during or after the sterilization process.

Declaration Of Conformity

The SteriTec Green Tape International conforms to EN ISO 11140-1 Type 1 Process Indicators.