

Wash-Checks® Cleaning Monitor

WC 101

Indications for Use

The Wash-Checks Disposable Cleaning Monitor is designed for monitoring cleaning efficiency during the wash process.

Instructions for Use

1. Insert a Wash-Checks monitor into a Wash-Checks Holder completely. The front side of the monitor with red test soil will be visible in the holder window.



2. Position Wash-Checks Holder face up inside a basket. Place basket on a shelf in the washer. Vary positions and shelves of Wash-Checks placement cycle to cycle.
3. Run a complete washer cycle.
We recommend monitoring every shelf on every washer each day for a machine release and using a minimum of one Wash-Checks per load for routine monitoring throughout the day.
4. At the end of the washing cycle, remove the Wash-Checks monitor from the Holder and read the result.
Caution: Wash-Checks Holder and monitor strip may still be hot! Remove with care.

Monitor Results Interpretation:



Untreated monitor or critical / poor result



*Insufficient result
This is an example of one type of fail*



Optimum result

5. You may also record the location of monitor placement in the washer on the Wash-Checks with a fine tip waterproof marker.



Place a hash mark or circle representing the position of the Wash-Checks on the location rectangle that simulates an imaginary washer basket. This allows for placement tracking.

Storage

- Store Wash-Checks monitors inside the protective foil bag provided at 40° - 86°F (4° - 30°C)
- Do not remove Wash-Checks monitors from foil bag until ready for processing
- Keep away from disinfectants and fumes

Expiry Date

The expiry date is printed on the product packaging. Do not use after expiration date listed on the packaging.