

Wash-Checks U Cleaning Monitor WC 108

Wash-Checks U is a single use synthetic soil test for routine monitoring of an ultrasonic wash process.

The test is inserted in a Wash-Checks Holder and then placed in the ultrasonic washer.



Key Features

- Tests effective cavitation, time, temperature and detergent
- No actual blood components contained in the test soil, removing potential for cross-contamination
- Reusable holder

Performance Characteristics

The synthetic test soil (blue square) simulates human blood and tissue. When placed in the holder, the test soil is fully exposed. The Wash-Checks Holder should be placed face up inside the ultrasonic washer. Run a cleaning cycle according to the manufacturer's recommendations. After the wash cycle is complete, results are displayed on the Wash-Checks U. The removal of all test soil indicates effective cleaning.

Physical Characteristics

Dimensions: 89 mm x 25 mm

Storage

The storage conditions are printed on the product packaging. Keep in the resealable bag provided

Shelf Life

24 months from date of manufacture. The expiry date is printed on the product packaging

SteriTec Part No.

WC 108, Wash-Checks U

Packaging

5 bags of 250

FTP-301 Rev. F



SteriTec Products Mfg. Co., Inc.,
Member of the Getinge Group
74 Inverness Drive East
Englewood, CO 80112 USA

www.getinge.com/steritec