

# Test Soil

## WC 120

Test Soil has been carefully designed to simulate the soiling that occurs during theatre use. The Test Soil is supplied in individual pots.



### Performance Characteristics

Water is added to the pot then shaken vigorously to form a paste which is then applied to a test load with the brush provided. After a drying period the test load should be cleaned using the normal procedures and inspected for residual soil. The bright red color of the Test Soil allows easy identification of areas that have not been properly cleaned.

### Physical Characteristics

- **Dimensions:** H 63mm x 43mm
- **Substrate Material:** Plastic vial with a lid with integrated brush

### Storage

The storage conditions are printed on the product packaging

### Shelf Life

The expiry date is printed on the product packaging

### Standards Classification

Consistent with BS2745:1993 specification for Washer-disinfectors for medical purposes. EN ISO 15883-5. HTM2030 Washer-disinfectors

### SteriTec Part No.

WC 120, Test Soil

### Packaging

4 boxes of 10 pots

FTP-WC120 Rev. B



SteriTec Products Mfg. Co., Inc.,  
Member of the Getinge Group  
74 Inverness Drive East  
Englewood, CO 80112 USA

[www.getinge.com/steritec](http://www.getinge.com/steritec)